



Accel[®]
PREvention^{™/MC}
Concentrate
Concentrated
One-Step
Surface
Cleaner &
Disinfectant



Tuberculocidal, Virucidal, Bactericidal, Fungicidal,
Broad-Spectrum Sanitizer

A broad spectrum, intermediate level concentrated
disinfectant based on proprietary AHP[®] technology
to deliver fast, effective cleaning performance.

Features

- Broad spectrum disinfectant concentrate
- Powered by AHP[®] technology
- No VOCs, no added fragrance and no NPEs
- Environmentally responsible-active ingredient breaks down to oxygen and water

Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus and Poliovirus
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 5 minutes at 1:40
- 30-second broad-spectrum sanitizer at 1:128
- Compatible with most hard, non-porous surfaces

Applications

- Clean and disinfect non-critical medical devices, equipment and non-porous hard surfaces



Accel® PREvention™/MC Concentrate

Concentrated One-Step Surface Cleaner & Disinfectant



Use Directions*

Preparation of Use-Solution

Ratio	Per Liter of Water	Per 32 oz. of Water
1:40	25 mL	0.8 oz.
1:128	8 mL	0.25 oz. (1/4 oz)

For use as a One-Step Surface Cleaner & Disinfectant

- Heavily soiled surfaces require cleaning prior to disinfection
- Dilute 1:40 and apply to surface with cloth or disposable wipe
- Let treated surface remain wet for 5 minutes
- Wipe dry with a cloth, sponge, mop or allow to air dry
- Prepare a fresh solution if the solution becomes visibly dirty or diluted

For use as a Non-Food Contact Sanitizer

- Dilute 1:128 and apply to surface
- Allow to remain wet for 30 seconds
- Wipe surface and let air dry

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey Care representative.

* Refer to product label for complete use instructions

5L Closed Loop Bottle Set Up Instructions

- Remove the screw cap from Accel PREvention Concentrate 5 L bottle
 - Affix the Accel Custom Dilution System mating cap* to the 5 L bottle (mating cap will attach to the insert inside the 5 L bottle to draw out the concentrate)
 - See Accel Custom Dilution System Instructions for Use for further dilution system instructions
- *Closed Loop Mating Cap included

Product Specifications

Description	
DIN	02448629
Certifications	EcoLogo
Color/Form	Colorless, liquid
pH	0.45 (Concentrate) 2.2 (Use Dilution 1:40)
Scent	Characteristic
Shelf Life	2 years (Concentrate), 30 days (Use Dilution)

Available Items

SKU Code	Description/Package Size	Dilution
100906586	4 X 4 L / 1.06 U.S. Gal. Containers	1:40, 1:128
100906588	2 X 5 L / 1.32 U.S Gal. Containers	1:40, 1:128
100940209	2 x 1.5 L RTD* / 50.7 oz.	1:40, 1:128

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at www.sealedair.com or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.

www.sealedair.com

© 2016 Sealed Air Corporation. All Rights Reserved. PSS90446 (16/295)